



QUICK START GUIDE VIO® 3


 This quick start guide is no substitute for briefing and user manual. This quick start guide does not include any safety notes. Make yourself familiar with the safety notes in the user manuals.

Necessary operating steps

1 Switch on the unit




2 Select program

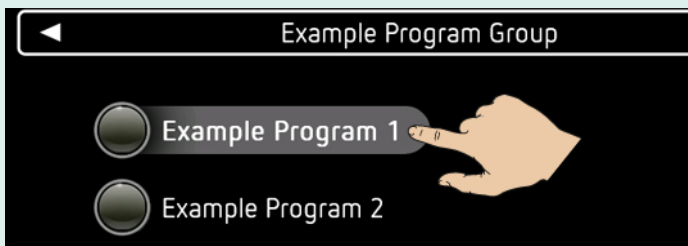

- Select a program without connected instrument cable. If an instrument cable has been already plugged in, disconnect it.

3 Apply and connect neutral electrode

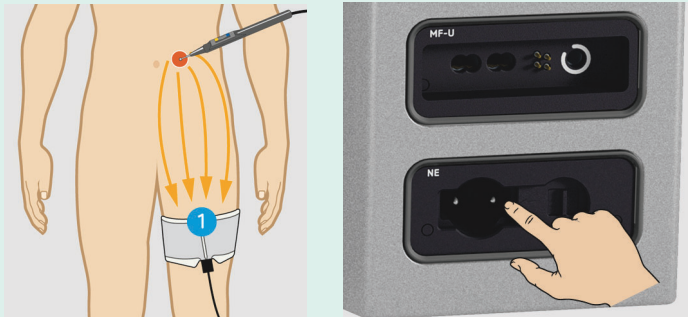

- Apply neutral electrode on appropriate location. The illustration is just an example. The long edge of the neutral electrode (1) must point to the procedural field.


4 Monitor neutral electrode




If the red neutral electrode symbol lights up, check the following:

- Is the neutral electrode correctly connected to the unit?
- Does the long edge of the neutral electrode point to the procedural field?
- Does the neutral electrode have sufficient contact to the skin?

5 Connecting the instrument

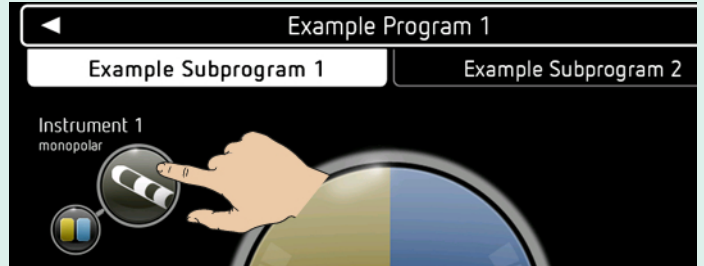


1. If subprograms are available, select the appropriate subprogram tab (operating step not shown).
2. Touch the instrument symbol.

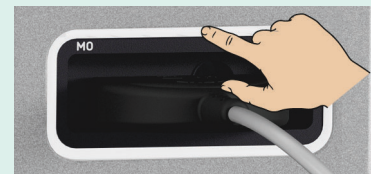


3. Connect the instrument to the socket with flashing socket frame.

6 Checking the instrument settings

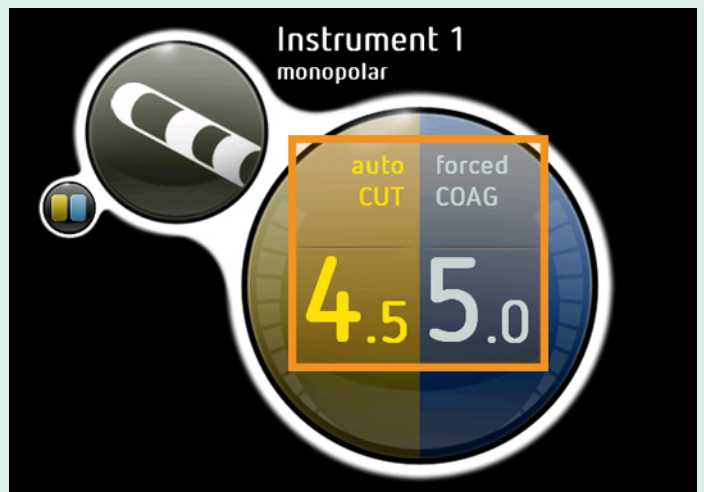


7 For APC applications, open argon bottle

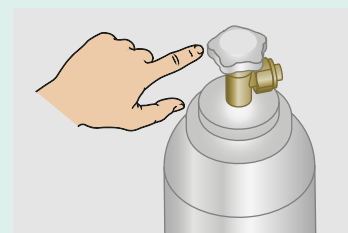


Optional operating steps

8 Changing mode

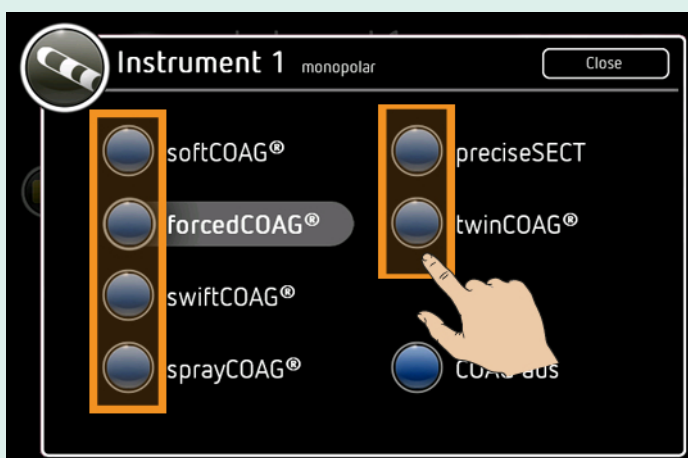

9 Changing effect

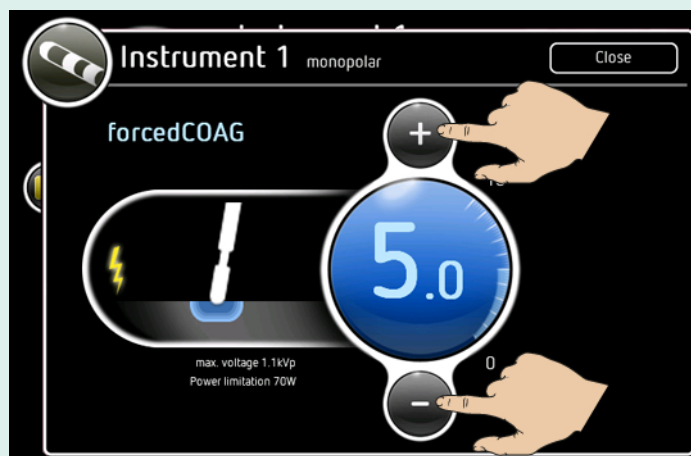

10 Assign footswitch, AUTO START, AUTO STOP

